

# AdVance™ XP

Male Sling System

**The next-generation  
solution for male stress  
urinary incontinence.**

Help your patients find happiness.

The world's best-selling male sling.<sup>1</sup>

Consider the AdVance XP Male Sling System to restore normalcy and renew confidence.

## Urologist benefits


- **Efficient procedure**
  - Chevron anchoring mechanism designed to increase acute holding power of the sling
  - Integrated permanent tensioning sutures ensure consistent support
  - Modified angle enhances trocar passage
  - Longer sling arms improve ease of use in larger patients
  - Twice the shelf life of first-generation sling<sup>1</sup>
- **Established outcomes**
  - More than 8 years of clinical use<sup>1</sup>
  - 90% success rate at 3 years in patients with mild to moderate incontinence<sup>2</sup>
  - More than 20,000 patients treated

## Patient benefits


- Minimally invasive procedures designed to allow most patients to resume normal, daily activities faster than with the previous generation device<sup>3</sup>
- Patients may be continent immediately following their procedure<sup>4</sup>
- Patient manual dexterity and mental acuity are not criteria for exclusion<sup>4</sup>
- Can be used in combination with any penile prosthesis<sup>5</sup>
- If needed, the transition to an AMS 800™ Urinary Control System does not usually require mesh excision<sup>6</sup>

# AdVance™ XP Male Sling System

## Mechanism of action

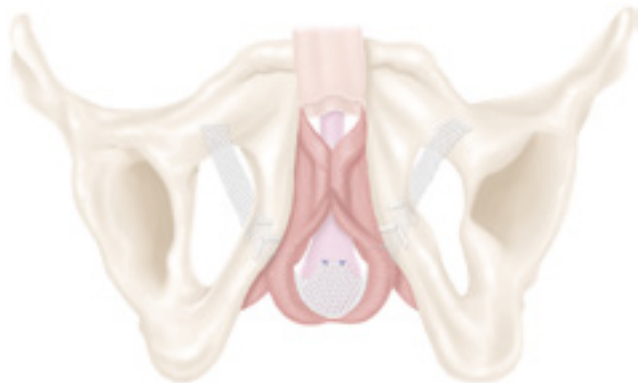

The sling is placed flat along the corpus spongiosum.

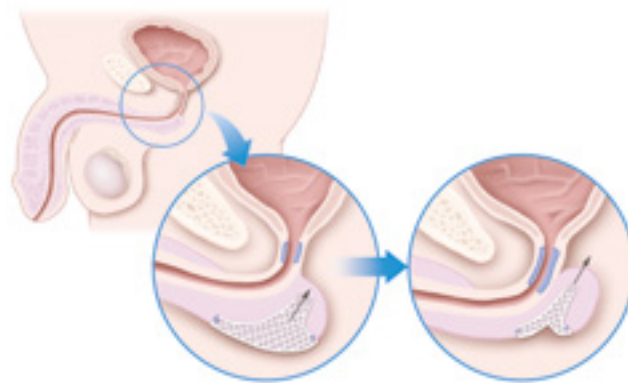

The AdVance XP Male Sling is then tensioned, thereby repositioning the bulbous urethra and resulting in an increased functional urethral length and a backdrop of support to the distal membranous (sphincteric) urethra during stress events.

## Patient risks

- Possible side effects could include pain, inflammation, urethral or tissue damage, bleeding and irritation at the wound site and urinary retention, which typically resolves within a few days<sup>3,7</sup>
- Contraindications include patients with urinary tract infections, blood coagulation disorders, compromised immune systems, renal insufficiencies or upper tract obstructions or upper urinary tract obstructions<sup>3</sup>
- Patients should abstain from heavy lifting, strenuous exercise and intercourse for a minimum of six weeks<sup>3</sup>

## AdVance XP Sling ordering information

	Product number
AdVance XP Male Sling Kit	720163-01

**“The modifications to AdVance XP — lengthening of the sling, trocar modifications and chevrons — will allow surgeons to treat more patients who are appropriate for a male sling and may be otherwise limited by their body habitus.”**

**Brian Christine, MD**  
Specialist in Prosthetic Urology and Men's Sexual Health, Urology Centers of Alabama, Birmingham, AL

1. Data on file with Boston Scientific.
2. Bauer RM, Grabbert MT, Klehr B, et al. 36-month data for the AdVance XP® male sling: results of a prospective multicenter study. *BJU Int.* 2017 Apr;119(4):626-30.
3. AdVance™ XP Male Sling System Instructions for Use. Boston Scientific Corporation. 2018.
4. Welk BK, Herschorn S. The male sling for post-prostatectomy urinary incontinence: a review of contemporary sling designs and outcomes. *BJU Int.* 2012 Feb;109(3):328-44.
5. McCammon K, Haab F. AdVance Male Sling: surgical technique and postoperative patient management. *Eur Urol Supplements.* 2011 Jul;10(4):395-400.
6. Ajay D, Zhang H, Gupta S, et al. The artificial urinary sphincter is superior to a secondary transobturator male sling in cases of a primary sling failure. *J Urol.* 2015 Oct;194(4):1038-42.
7. DeRidder D, Webster G. Clinical overview of the AdVance Male Sling in post-prostatectomy incontinence. *Eur Urol Supplements.* 2011 Jul;10(4):401-6.

All trademarks are the property of their respective owners.

CAUTION: The law restricts these devices to sale by or on the order of a physician. Indications, contraindications, warnings and instructions for use can be found in the product labellingsupplied with each device. Information for use only in countries with applicable health authority registrations. This material not intended for use in France.

Products shown for INFORMATION purposes only and may not be approved or for sale in certain countries. Please check availability with your local sales representative or customer service.

MH-580120-AA Printed in Germany by medicalvision.

**Boston Scientific**  
Advancing science for life™

[www.bostonscientific.eu](http://www.bostonscientific.eu)

©2018 Boston Scientific Corporation or its affiliates. All rights reserved.

DINURO2367EA