


## Digitally read, unambiguous results help you move forward with more confidence

The BD Veritor™ Plus System delivers clear, digitally displayed results for a menu of respiratory tract infections.

Assay	Time to Result	Analyte	Reference Method	PPA	NPA
SARS-CoV-2 <sup>1</sup>	15 minutes	SARS-CoV-2	RT-PCR	<b>91.1%</b> (95% CI, 80.7%-96.1%)	<b>99.6%</b> (95% CI, 97.9%-99.9%)
SARS-CoV-2 & Flu A+B <sup>2</sup>	15 minutes	SARS-CoV-2	RT-PCR	<b>86.7%</b> (95% CI, 75.8%-93.1%)	<b>99.5%</b> (95% CI, 97.4%-99.9%)
		Influenza A		<b>82.7%</b> (95% CI, 74.9%-88.5%)	<b>97.5%</b> (95% CI, 95.7%-98.5%)
		Influenza B		<b>80.7%</b> (95% CI, 70.3%-88.1%)	<b>98.2%</b> (95% CI, 95.7%-99.3%)
Flu A+B <sup>3</sup>	10 minutes	Influenza A	PCR	<b>83.6%</b> (95% CI, 76.1%-89.1%)	<b>97.5%</b> (95% CI, 95.7%-98.5%)
		Influenza B		<b>81.3%</b> (95% CI, 71.1%-88.5%)	<b>98.2%</b> (95% CI, 95.7%-99.3%)
RSV <sup>4</sup>	10 minutes	RSV	PCR	<b>81.6%</b> (95% CI, 75.2%, 86.6%)	<b>99.1%</b> (95% CI, 97.5%-99.7%)
Group A Strep <sup>5</sup>	5 minutes	Group A Strep	Bacterial Culture	<b>Sensitivity 96.6%</b> (95% CI, 92.4%-98.6%)	<b>Specificity 95.5%</b> (95% CI, 93.6%-96.9%)

Results shown for Point of Care test kits. Laboratory test kits are also available for Flu A+B and RSV.

Reliable results in a timely manner allow you to quickly diagnose and help improve care for your patients<sup>6,7</sup>

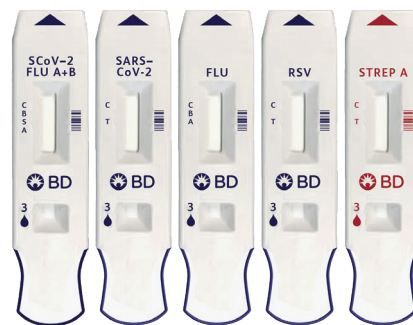

CI=confidence interval; NPA=negative percent agreement; PCR=polymerase chain reaction; PPA=positive percent agreement; RSV=respiratory syncytial virus; RT-PCR=reverse transcriptase–polymerase chain reaction; SARS-CoV-2=severe acute respiratory syndrome coronavirus 2.

**References:** 1. BD Veritor™ System for Rapid Detection of SARS-CoV-2. Package insert, 500050809. Becton, Dickinson and Company. 2. BD Veritor™ System for Rapid Detection of SARS-CoV-2 & Flu A+B. Package insert, 500059135. Becton, Dickinson and Company. 3. BD Veritor™ System for Rapid Detection of Flu A+B. Package insert, 8087667. Becton, Dickinson and Company. 4. BD Veritor™ System for Rapid Detection of Respiratory Syncytial Virus (RSV). Package insert, 8086098. Becton, Dickinson and Company. 5. BD Veritor™ System for Rapid Detection of Group A Strep. Package insert, 8087675. Becton, Dickinson and Company. 6. Koonin LM, Patel A. Timely antiviral administration during an influenza pandemic: key components. *Am J Public Health*. 2018;108(Suppl 3):s215-s220. 7. Huang G, Gong T, Wang G, et al. Timely diagnosis and treatment shortens the time to resolution of coronavirus disease (COVID-19) pneumonia and lowers the highest and last CT scores from sequential chest CT. *AJR Am J Roentgenol*. 2020;215(2):367-373.

BD - Europe, Terre Bonne Park - A4, Route de Crassier 17, 1262 Eysins, Switzerland

[bd.com/VeritorSystem-EU](http://bd.com/VeritorSystem-EU)

